


Program Basics – Moving around in the Data Input Panel

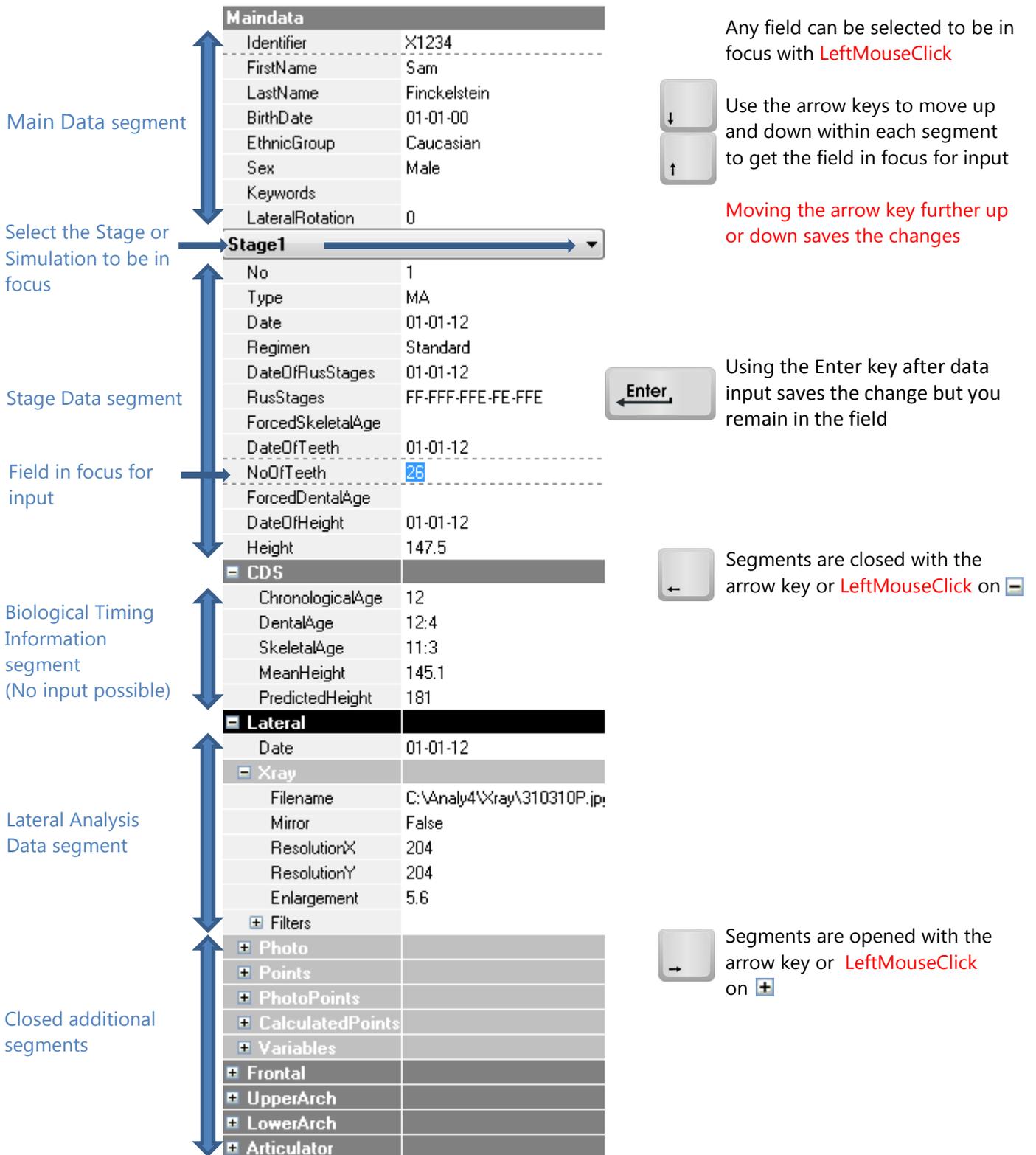

The screenshot shows the TIOPS Data Input Panel with several segments and navigation instructions:

- Main Data segment:** Contains fields like Identifier (X1234), FirstName (Sam), LastName (Finckelstein), BirthDate (01-01-00), EthnicGroup (Caucasian), Sex (Male), Keywords, and LateralRotation (0).
- Select the Stage or Simulation to be in focus:** A dropdown menu currently showing "Stage1".
- Stage Data segment:** Contains fields like No (1), Type (MA), Date (01-01-12), Regimen (Standard), DateOfRusStages (01-01-12), RusStages (FF-FFF-FFE-FE-FFE), ForcedSkeletalAge, DateOfTeeth (01-01-12), NoOfTeeth (26), ForcedDentalAge, DateOfHeight (01-01-12), and Height (147.5).
- Field in focus for input:** The "NoOfTeeth" field is highlighted with a blue selection box.
- Biological Timing Information segment (No input possible):** Contains fields like ChronologicalAge (12), DentalAge (12:4), SkeletalAge (11:3), MeanHeight (145.1), and PredictedHeight (181).
- Lateral Analysis Data segment:** Contains fields like Date (01-01-12), Xray (with sub-fields: Filename, Mirror, ResolutionX, ResolutionY, Enlargement), Filters, Photo, Points, PhotoPoints, CalculatedPoints, Variables, Frontal, UpperArch, LowerArch, and Articulator.
- Closed additional segments:** A list of segments like Photo, Points, PhotoPoints, CalculatedPoints, Variables, Frontal, UpperArch, LowerArch, and Articulator, each with a plus sign icon.

Navigation instructions:

- Use the arrow keys to move up and down within each segment to get the field in focus for input.
- Moving the arrow key further up or down saves the changes.
- Using the Enter key after data input saves the change but you remain in the field.
- Segments are closed with the arrow key or LeftMouseButton on the minus sign icon.
- Segments are opened with the arrow key or LeftMouseButton on the plus sign icon.

Note: You can use either , or . as a decimal separator

Note 2: Any item in a field containing a drop down list can be selected by clicking on the item. You can also select the item without opening the list by typing the first letter. Items with a common initial letter are chosen by typing the first two letters.

