



Program Basics – Repositioning Images in the Picture Windows

The following Mouse Functions can be used both in the Main PictureWindow and the Jaw- and Occlusogram Window.


1. To **translate** the whole screen image:
Position the cursor in the window.
Press <Ctrl> and <LeftMouse> and drag while holding <LeftMouse> down.

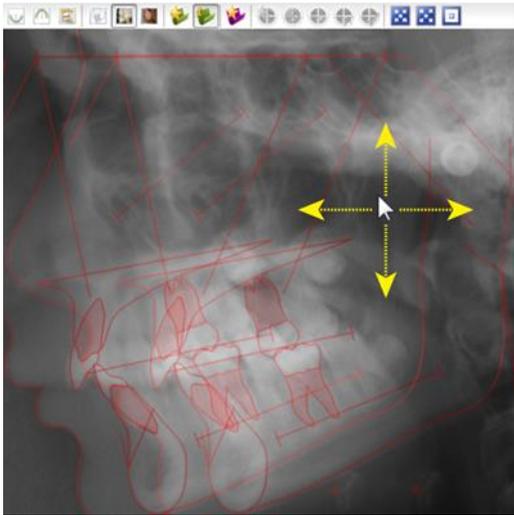


2. To **rotate** the whole screen image:
Position the cursor in the window.
Press <Alt> and <LeftMouse> and drag while holding <LeftMouse> down.

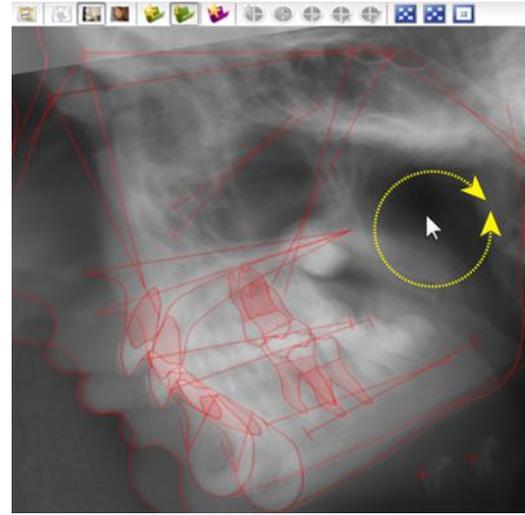

3. To **zoom in** on part of an image:
Hold <RightMouse> and drag a rectangle

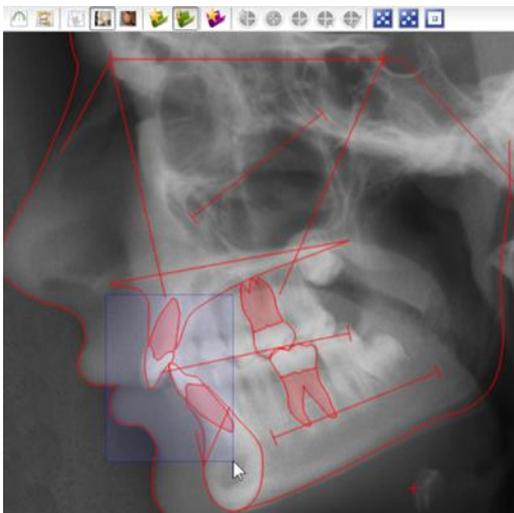

Release <RightMouse>

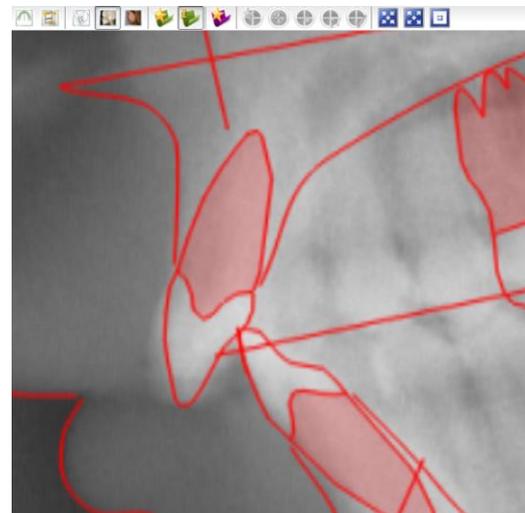

Note:
The smaller the rectangle the greater the zoom

4. To **reset** the an image to its original size:
Press <RightMouse>
Pressing <RightMouse> repeatedly will toggle between 1:1 size and 75% size

The following **icons** can only be used in the Main PictureWindow:



- Zoom In** Pressing the icon or hitting <F10> enlarges the picture in 10% steps
- Zoom Out** Pressing the icon or hitting <F11> reduces the picture size in 10% steps
- Center** Pressing the icon or hitting <F12> will center the picture in the window